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- ⇒ **E.-J. Fiskerstrand, Trondheim/Norway (LasersSurgMed 2003;32:399-404): Hair Removal with Long Pulsed Diode Lasers: A Comparison Between Two Systems With Different Pulse Structures.**
  - 29 patients, skin type II – III, 3 treatments, light brown till dark brown hair, upper lip + 3 patients bikini area
  - half side comparison LightSheer (9x9) vs. MeDioStar (12 mm), fluence 35 J/cm<sup>2</sup>
  - same effectiveness (after 3 treatments 49 % reduction with MeDioStar vs. 48% LightSheer)
  - no pigmentation shift, no scars, less erythema and burned hair with MeDioStar
- ⇒ **A.J. Nemeth, Clearwater/Florida (EADV 2004): „Impressive laser hair removal with an 810 nm powerpulsed diode laser: safety and efficacy.”**
  - 200 patients, 978 sessions in total, fluence 10 – 36 J/cm<sup>2</sup>, 12 mm spot, skin type I – VI, 84 % II – III
  - 3 – 8 treatments (Ø 4,9), 64 % hair reduction after 3rd treatment
  - after 5 treatments (0,5 %) transient hyper pigmentation
- ⇒ **N.S. Sadick, New York/USA (DermSurg 2003;29:30-34): „The Use of a New Diode Laser for Hair Removal.”**
  - 24 patients, skin type I – IV, 3 treatments, axilla, bikini area, color of hair: light – dark brown
  - 70 % clearing 6 month after 3rd treatment, 1 transient hyper pigmentation for 2 weeks
- ⇒ **T. Ilknur, Uni Izmir/Türkei (LasersMedScience 2010; Okt.): „Effects of the 810-nm diode laser on hair and on the biophys. properties of skin”**
  - 31 patients, skin type II – IV, Axilla
  - comparison of bio physics. Skin characteristics before/after 1 treatments vs. untreated
  - effectiveness: 48 % hair reduction 6 weeks after 1 treatment, statistically significant reduction of hair thickness
  - erythema index up to 2 weeks changed after treatment
  - no changes: melanin index, water content of the skin, pH-value, sebum secretion
- ⇒ **A.J. Nemeth (2005)**
  - Half side comparison of wavelengths 810 vs. 940 nm, 25 patients, 88 sessions in total
  - skin type II – IV, fluence 11 – 32 J/cm<sup>2</sup>, 12 mm spot
  - same effectiveness, no side effects with 940 nm, 2 with 810 nm (1 crusts, 1 temporary hyper pigmentation)
- ⇒ **A. Steps (2011): „Diode Laser Hair Removal: a Study with a new high power system”**
  - comparison MeDioStar NeXT and MeDioStar miXT, 12 patients, skin type I – III, Axilla
  - MeDioStar NeXT: 49 % hair reduction after 3 treatments (17 – 91 %)
  - result slightly better than with MeDioStar miXT
- ⇒ **Dr. Landwehr (2012): „Monitoring study of permanent hair removal with high power diode laser MeDioStar NeXT carried out”**
  - MeDioStar NeXT: 27 patients, 157 treatments
  - more than 70 % of the patients were satisfied or really satisfied
  - the SmoothPulse mode is considerably more comfortable for the patient at same effectiveness

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- ⇒ **V. Tsepkoenko, Odessa/Ukraine (2003)**  
106 patients, vessel diameters up to 3 mm, fluence 80 – 160 J/cm<sup>2</sup>, 6 mm spot size  
75 % of vessels disappeared completely, 20 % clearly better, 4 % better, 1 % unsatisfied
- ⇒ **K. Kykalova, Prag/Czech (2011)**  
24 patients, vessel diameter 1 to 4mm, fluence 90 – 100 J/cm<sup>2</sup>, 2 – 5 treatments  
(used to treat with higher parameters, but same results with less side effects)  
results up to Ø 2 mm: 70 % disappeared completely, Ø 2 – 3 mm : 50 %, Ø 4 mm: 30 % with 3 treatments
- ⇒ **A. Steps, Germany**  
21 patients, MeDioStar NeXT vs. MeDioStar miXT: shorter pulses at same fluence possible, less side effects with MeDioStar NeXT

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- ⇒ **C. Passos-Pereira (Dermaforum Sept. 2003)**  
report of experience treatments Acne inversa, 20 patients, fluence 30 – 40 J/cm<sup>2</sup>, 12 mm spot, treatment like epilation  
appearance: from recurrent abscess building to monthly abscess splitting under a general anesthetic, all patients responded to the therapy  
favorable treatment at primary stage of the pimple or abscess without scar formation, clear improvement of results and general health in case of advanced course, renunciation of primary radical OP with covering of defect and restriction on minimal surgery possible, which improves the patient's situation; additional finding reduction of odor production

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- ⇒ **Dr. Dai et al., Taipei/Taiwan (2008)**  
Objective measurement of skin quality improvement after rejuvenation treatment, fluence 30 – 40 J/cm<sup>2</sup>, pulse duration 300 – 400 ms, 12 mm spot, 3 passes, 1 treatment, measurement of wrinkles, pore size, texture and age spots in % pre and post 1 treatment  
improvement of texture: 22 – 31 %, wrinkles: 33 – 39 %, pore size 10 – 19 %, spots: 15 % in % pre/post 1 treatment
- ⇒ **S. McCoy, Adelaide/Australia**  
12 patients with numerous Lentiginos, fluence 25 – 50 J/cm<sup>2</sup>, spot initially 12 mm, afterwards 6 mm (shorter pulses)  
results less dependent on fluence, but strong coloring of pigmentation (dark better)
- ⇒ **Institute of Dermatology, Ministry of Public Health, Bangkok, Thailand (J Cosmet Dermatol. 2015 Dec;14(4):E7-14. Epub 2015 Jul 15.)**  
Significant improvement was observed with the Cutometer F3 and R7 parameters at 1 and 3 months after complete treatment, respectively. Physician assessment showed significant improvement in the laxity scale at 1 and 6 months after treatment. 98 % of the patients were satisfied with the treatments.